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Rollator vital classic 54

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Please read these notes carefully before use. Be sure to follow the instructions and keep them in a safe place! If this product is to be used by other persons, they must be fully informed of these safety warnings before use.

Indication/Contraindication

The rollator is a mobility aid for people with mobility restrictions, e.g. people with a disability that makes walking difficult or who cannot walk for long, providing them with the opportunity to go further afield. Staying on the move with a rollator not only serves to improve circulation, coordination and balance, but it also enhances the user's well-being.

This rollator is designed both for use in enclosed areas and in the open. It is equipped with a removable bag. The rollator provides a seat for short rests. It can be folded easily for transportation, thus saving space. The rollator should not be used by persons with perception impairments, serious problems with their balance or an inability to sit. The user must have adequate perception skills and adequate physical ability to be able to steer a rollator.

⚠ Safety Indications ⚠


- Talk to your doctor or specialist retailer to see whether this Rollator is optimally suitable for you and your needs.
- Talk to your doctor or specialist retailer to see whether this rollator is optimally suitable for you and your needs.
- Check that the rollator is in perfect condition every time before you use it. Check the following components and make sure:
 - all screws, nuts and other parts are firmly tightened
 - all wheel surfaces are intact
 - the height adjustment screws are firmly tightened
 - the brakes function both as parking brakes and hand-brakes
 - there are no visible signs of wear on the frames, welded joints and plastic elements
 - wheels and wheel forks should be free of dirt and turn smoothly.
- Ensure that the rollator is correctly engaged before every use.
- Always activate the parking brake when you stop or park the rollator. Always use the hand brakes slowly, evenly and both together.
- The braking effect can be reduced if the tyre surface is worn. Inspect the braking action of the brakes regularly.
- The rollator should only be used on firm and even surfaces. There is a risk of slipping on dirty, wet, frosty or snow-covered ground! The braking effect is also limited. Please adapt the way you move and walk to these dangers.
- On smooth surfaces such as tiles, stone flooring, PVC, etc. the wheels can block when braking under certain circumstances. Therefore please check the braking performance of your rollator carefully on these floors and adjust the way you move and walk accordingly. If you wish to sit on the rollator you should also make sure that the wheels have a sufficient hold on the surface when the brakes are applied and that the rollator cannot slide away.

Do not use a damaged or defective rollator!

- The rollator should not be used as a support when getting up from a crouching or sitting position! It should only be used for its intended purpose and should not be used for transporting objects over 5 kg or persons.
- Ensure that the handles are adjusted to the correct height for the user in order to avoid improper weight load or falls. We recommend you walk between the handles. Always hold both handles; otherwise you can lose your balance.
- Make sure that your hands or clothes do not come into contact with the moving parts of the rollator! Be particularly careful when folding and unfolding the rollator. This applies in particular to hinges under the seat or in the middle of the cross brace.
- The rollator should be used on the pavement. The rollator should not be used in moving traffic. For your personal safety on the roads it is recommended that you also wear your own reflectors as well as those fixed to the rollator.
- Do not sit on the rollator when travelling on public transport. Park the rollator with the brakes on so that it cannot fall over or roll away. Look for a seat close by.
- Be careful near children. Children should not play with the rollator!
- The maximum permissible loading for the rollator is 150 kg. This includes the maximum load in the bag of 5 kg.
- Do not hang any handbags or carrier bags on the handles, the balance of the rollator can be impaired if you do.
- Do not carry out any adjustments or modifications, which impair the safety of the design. Parts such as wheels, handles or brake cables may only be replaced by a specialist retailer.
- Only use accessories approved by the manufacturer. Please also read the instructions for use for each accessory.
- Do not carry or pull the rollator by the seat, the strap for folding or the brake cables.
- In strong sunlight parts of the rollator, e.g. the handles and seat, can get hot. Please ensure that you cover your rollator up or leave it in a shady place.
- Please prevent skin contact with the rollator if the rollator has been in cold temperature for a long period of time in order to prevent freezing.
- Keep the rollator away from naked flames.
- Please prevent running into door frames, furniture, etc. with the rollator and check the secure position of the rollator when parked. The frame, connection elements and welding seams may be damaged if the rollator tips over.
- Please also prevent placing stress on the rollator when folded together! This could damage the frame, connection elements and welding seams.
- Do not use the rollator in stairs or escalators.
- If you have any queries, please refer to your dealer.

Remove all packaging material. Check the product has not been damaged in transit before using it for the first time. Pay particular attention to ensuring that the wheels and wheel suspension, screw and plastic connections and brake cables are intact. The rollator is delivered pre-assembled. Before using for the first time, check that the brakes work properly; they may need to be readjusted. The hand grips must be adjusted to the appropriate height for the user. Please dispose of the packaging material correctly.

Contents/Description of the parts:

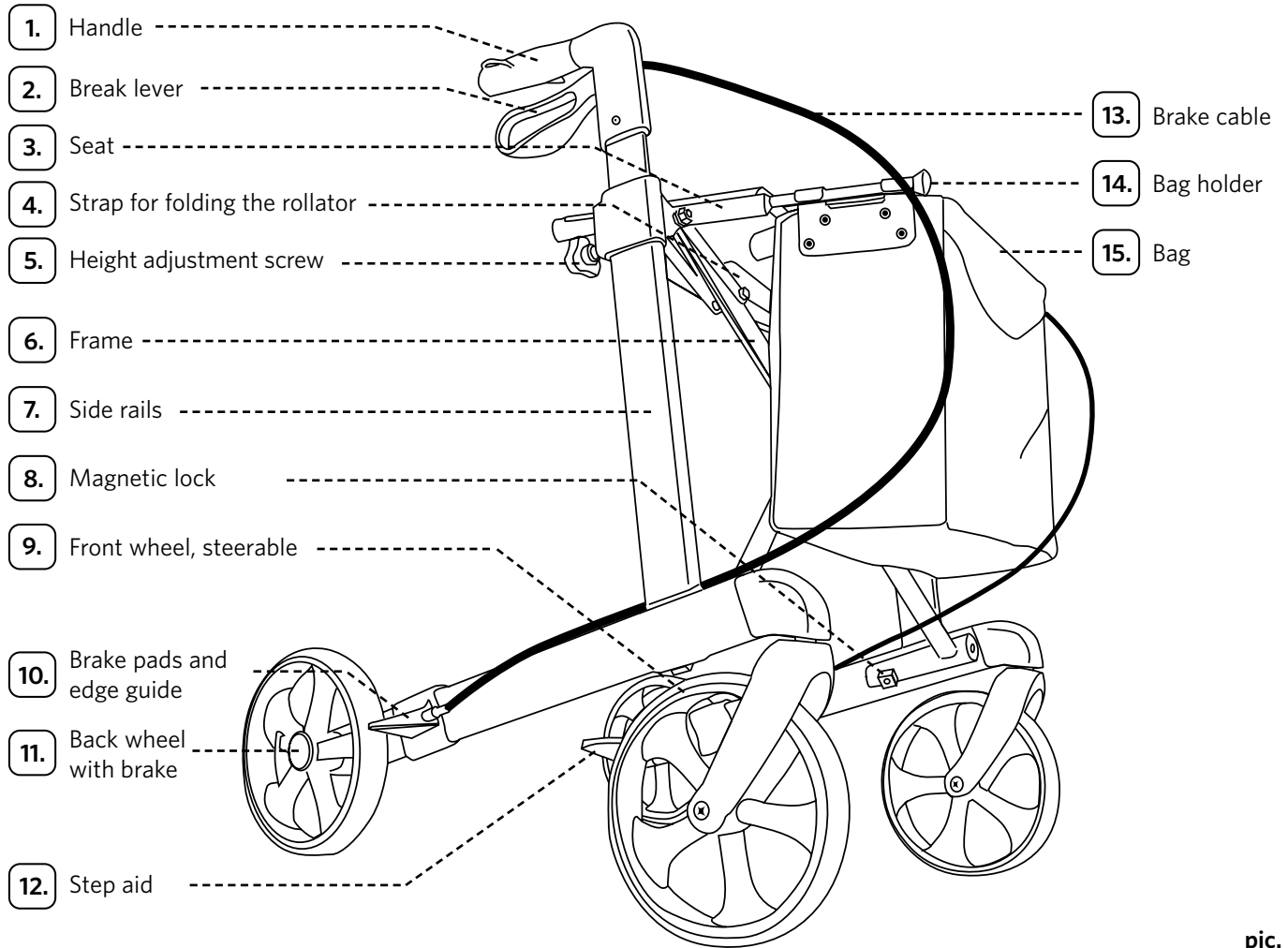pic. 1

Assembly

Opening the rollator

1. Pull the handles apart (pic. 2).
2. Push the side rails downwards until they engage. the rollator is now fully opened (pic. 3).
3. Make sure the rollator is engaged before you use it.

pic. 2

pic. 3

Folding up the rollator

1. Fold the rollator up by pulling the strap upwards (pic. 4).
2. Push the handles inwards and the rollator together (pic. 5). The fully folded rollator is locked with magnets (pic. 6).

pic. 4

pic. 5

pic. 6

Height adjustment of the handles (pic. 7)

Adjust the height of the handles so that the user can walk upright, without bending over. It is recommended that the user stands upright with arms at their sides between the handles. The grips should be adjusted to the height of their wrists.

1. Loosen the height adjustment screw (pic. 1, No. 5).
2. Move the handle to the desired height.
3. Tighten the height adjustment screw (pic. 1, No. 5). Adjust both handles in the same height. The printed numbers on the inside of the push handles help you easily adjust your personal grip level again (pic. 8).

pic. 7

pic. 8

Brake (pic. 9)

The built-in brake functions as an operating brake for slowing down when moving. The parking brake can be used when the rollator is stopped.

1. Speed is reduced by pulling the brake lever upwards (operating brake).
2. The parking brake is activated by pressing the brake lever down until it engages.
3. The parking brake is released from its locked position by pulling the brake lever upwards.
4. The rollator should not be pushed while the operating or parking brakes have been applied. If the rollator is pushed while the operating or parking brakes have been applied, the tyres and brakes may be subject to excessive wear and deteriorate.
5. Regularly check that the brakes work satisfactorily. We recommend checking before every use. The rollator must not be used with damaged brakes or damaged wheels.


pic. 9

Brake adjustment (pic. 10)

If the braking force is decreasing due to wear on the wheels, the brake shoe can be adjusted. The brakes must be adjusted so that both wheels brake evenly. The distance between the brake pad and the wheel surface should be about 2-3,5 mm. After replacing wheels, the brake setting must be checked again and if necessary, readjusted.

1. Loosen the locking nut.
2. Adjust the braking force by turning the brake cable spanner inwards or outwards.
3. Check the braking function by turning the wheels and operating the brake levers. The wheels should turn freely but stop immediately when the brakes are applied. The wheels must not be able to move when the brakes are applied.
4. Tighten the locking nut firmly again.
5. If you are unsure about the correct adjustment of the brakes, please contact your specialist retailer.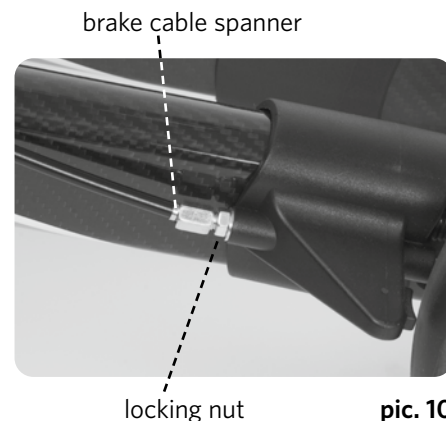


pic. 10

Seat

Activate the parking brake before you sit on the seat. Always sit fully on the seat with your back in the direction of travel. Do not lean against the seat! Do not lean over the sides or backwards while sitting on the rollator. That could impair the stability of the rollator. Do not use the seat on a sloping surface. Ensure that the front wheels are parallel to the direction of walking. The seat has been tested for a user weight of up to 150 kg.

Overcoming obstacles with the help of the step assistant

The step assistant is designed to help overcome shallow steps and kerbs. It is already mounted on both sides of the frame between the rear wheels. When you want to overcome an obstacle, move the rollator towards the obstacle as straight as you can until the two front wheels are parallel directly in front of the obstacle. Now press the step assistant down carefully with your right or left foot (pic. 11) and pull both push handles towards your body at the same time until the front wheels reach the top edge of the obstacle. Then push the front wheels of the rollator forwards onto the obstacle until the rear wheels also reach the obstacle. Now raise the rollator until it is fully on top of the obstacle. Now step up onto the obstacle yourself. Practise using the step assistant in a relaxed atmosphere with another person until you feel safe and secure using the function. Ensure you can stand safely when you use the step assistant.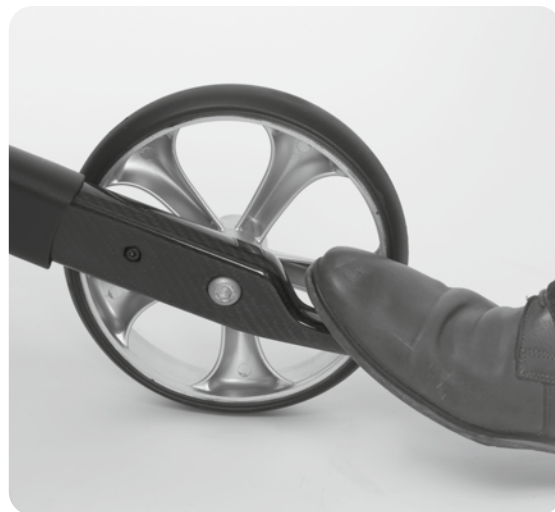


pic. 11

Care and Maintenance

Rollator

The rollator may only be used as a walking aid. Keep the rollator free from dirt, so that it remains functional. The rollator can be cleaned with a neutral cleaning agent and a cloth and then rinsed and dried.

Dry the rollator thoroughly after cleaning or exposure to rain. The brake pads and springs should also be cleaned regularly. The rollator should only be disinfected by a specialist. The product can be disinfected with a disinfectant with 70-80 percent ethanol content. Disinfectants containing chlorine or phenol should not be used. Do not use caustic solvents. Do not use the rollator in temperatures below -35 °C or above +60 °C.

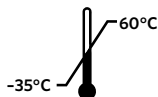Bag

Small items can be carried conveniently in the bag. The inside pockets are suitable for storing items like keys or your purse or wallet. The bag is designed to carry up to 5 kg. It can be hung up and removed again easily using the integrated handles.

Please find below a maintenance table and hygiene plan for the regular care and maintenance of this device:

Maintenance Table	
When	
Before every re-use	
What	Caution
Check general condition Visually inspect all frame parts including welded joints for deformation, damage, wear or corrosion	If you find any defects, replace the affected parts of the device.
Check hand grips visually and by attempting to turn them	If you find that the hand grips are damaged or no longer firmly connected to the frame, they must be replaced.
Check the tyre tread and ball bearings	If you find that the wheels are worn or cracked, they must be replaced.
Check the folding lock	If the folding mechanism no longer works properly, it must be repaired or replaced.
Check the connecting elements for tightness	If you find that screws, nuts, rivets, etc. are loose or missing they must be tightened, fitted or replaced.
Check the braking system is working satisfactorily Apply the operating brake until it reaches the stopping point. The wheels must not be able to turn now when pushed hard. Apply the parking brake by pressing the brake levers down until it engages. The wheels must not be able to turn now when pushed hard.	If the brakes are defective, the brake slack, brake cables, brake shoes and wheels must be checked and the defective part of the device must be replaced if necessary.

Send your rollator for regular checks to your dealer. He will be able to detect any wear and tear and replace parts, if necessary. If you find the brake action to be faulty in any way, please contact your dealer immediately.

Hygiene Plan

Before each re-use or before each patient change, the rollator is to be disinfected and cleaned.

Disinfection/cleaning

1. Before preparing the rollator, put on protective gloves and if necessary, protective clothing (disposable apron).
2. Always disinfect first, then clean. Exception: Severe contamination must be removed before disinfection.
3. Wipe or spray the rollator with a disinfectant listed by the German Society for Hygiene and Microbiology (DGHM) or the German Association for Applied Hygiene (VAH) (taking care to follow the manufacturer's recommended contact time). Disinfection by wiping is generally preferable to spraying. After the contact time for the disinfectant has elapsed, dry the rollator off with a clean, dry cloth.
4. To clean, use water and a mild detergent. No water must be allowed to enter the tubes. Do not use caustic or bleaching cleaning agents.

After cleaning, dry the rollator off with a clean, dry cloth.

Re-use

The product can be re-used. It must be cleaned and disinfected according to instructions. It must be inspected by specialist shops for any faults and then authorised before it can be used again. Please order our hygiene and maintenance guidelines.

Storage

Store the rollator light and dust-protected in a dry place. Do not store near a heat source. ☂️ ❄️
Protect the rollator from dirt and contamination after treatment or if it will not be used for a long period of time.

Disposal

Do not dispose of the rollator in the normal household rubbish. Ask your local recycling authority how to dispose of it properly and in an environmentally friendly way.

Technical Details

Material:

- Frame: Aluminium with plastic-segments
- Wheels: Thermoplastic rubber (TPR)

Rollator vital 62

- Dimensions: 60 x 66 x 74-101,5 cm (W x L x H)
- Folding measure: 22 x 66 x 74-101,5 cm
- Wheels: 20,2 x 3,5 cm
- Seat level: 62 cm
- Distance between handles: 44 cm
- Weight: 6,8 kg + bag (300 g)
- Turning circle: 84,5 cm
- Recommended body size: 170-210 cm
- Max. loading capacity: 150 kg

Bag

- Material: 100 % Polyester
- Dimensions: 32 x 15 x 28 cm (W x L x H)
- Permissible weight: max. 5 kg

Rollator vital 54

- Dimensions: 60 x 66 x 66-85 cm (W x L x H)
- Folding measure: 22 x 66 x 66-85 cm
- Wheels: 20,2 x 3,5 cm
- Seat level: 54 cm
- Distance between handles: 44 cm
- Weight: 6,64 kg + bag (300 g)
- Turning circle: 84,5 cm
- Recommended body size: 150-180 cm
- Max. loading capacity: 150 kg

Product Warranty

In addition to statutory warranty claims, we offer an eight-year warranty for this Rollator vital to private customers on the basis of the following warranty conditions:

We guarantee private customers that the device supplied to them will be free from material, workmanship or manufacturing defects for a period of eight years after purchase (based on the document purchase date).

We will rectify any defects at our discretion by replacing a new or refurbished rollator or by repair or by supplying new or refurbished parts.

Any other claims by the end customer against us, in particular claims for damages, which go beyond the statutory warranty claims, are excluded.

The warranty claims will not be accepted, in particular,

- in the case of normal signs of use on the rollator and/or its accessories such as the mesh seat, brake cables, wheels, hand grips, etc.;
- if the defects have been caused by abnormal use that does not comply with the specifications (according to the instructions for use and cleaning instructions);
- if the rollator shows characteristics indicating repairs or other interference by unauthorised specialist retailers;
- if accessories from other manufacturers have been installed in or fitted onto the rollator without obtaining prior written consent from us;
- if the type plate with the serial number has been disfigured or obliterated.

Further contractual or statutory rights of the private customer are not affected by the warranty. The warranty only applies to a rollator that has been purchased. The warranty period will not be extended if a replacement rollator or spare parts are supplied.

In the event of a warranty claim, the private customer must contact their specialist retailer from whom they purchased the rollator with proof of purchase or contact us by telephone on the free service number +49 800 8744874 (Mondays to Fridays 8.00 am – 5.30 pm).

This device complies with Regulation (EU) 2017/745 on medical devices and bears a CE marking.

If a serious incident occurs involving this medical device, contact the manufacturer of this device at info@russka.de as well as the competent authority, the Medicines & Healthcare products Regulatory Agency MHRA, www.gov.uk/government/organisations/medicines-and-healthcare-products-regulatory-agency.

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